


## Quick Start Set-up:

- Remove system from the box by lifting the top handles. Place on sturdy table. Remove the plastic covering from the A3500 system. Remove accessory box, funnel and power cord and store box for future use.
- Take a moment to review the A3500 photo and Parts Description.
- Choose the disposable plastic tip size (S, M, L) which best suits the treatment to be performed, e.g. small/medium tip for face, large tip for chest/back. Place the plastic tip on the top of the handpiece using a twisting motion while applying downward pressure until the pin on the handpiece fits into the groove on the plastic tip.
- Fill the clean container (the one without the filter) with crystals to the Full Line. Crystals should not be below the 450 mark on the container.
- Turn on fan switch located in the back left side of the system.
- Turn on system switch on front panel.
- Please refer to Operator Manual for treatment procedure.

## A3500 PowerPeel Microdermabrasion System

### Parts Description (on back of page)

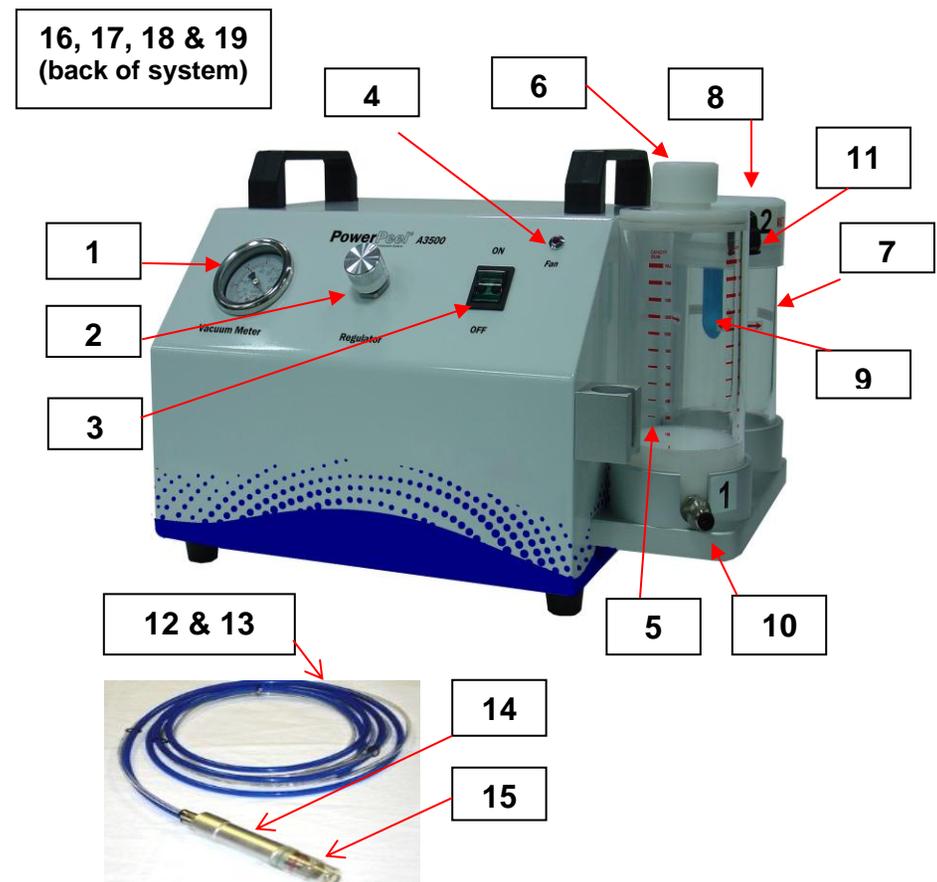

# A3500 PowerPeel Microdermabrasion System

## Parts Description

1)	Vacuum Meter	Indicates the power setting for the vacuum flow (suction).	13)	Crystal Recovery Tube (large tube)	One end of the tubing connects to the elbow and the other end connects to the handpiece.
2)	Regulator Knob	This regulates the power setting for the vacuum flow (suction).	14)	Operator Handpiece	Applicator that vacuums crystals across the surface of the skin.
3)	System Switch	Located on the front of the A3500.	15)	Disposal Plastic Tip	A plastic tip that is placed at the end of the handpiece to direct crystal flow.
4)	Fan Light	Light indicating fan is activated.	16)	Fan Switch	Located in left rear corner of system. Turn the switch to the "on" position when starting the microdermabrasion system. This acts as a cooling system for the machine (not visible in photo).
5)	Clean Crystal Container	Houses the clean crystals.	17)	Power Cord Plug	Located in left rear corner of system. Plugs into 120 Volt wall outlet (not visible in photo).
6)	Clean Container Cap	Clean Container Cap is easily removed by lifting up on the cap to add crystal to the container. Place cap back onto container and push down to secure seal.	18)	Fuse Housing	Located in center rear of system. Contains two 5 AMP fuses (not visible in photo).
7)	Waste Crystal Container	Houses the used (waste) crystals.	19)	Serial Number Plate	Manufacturer's identifying serial number located in center rear of cabinet (not visible in photo).
8)	Waste Container Lid	Waste Container Lid is easily removed by lifting up on the cap to dispose of used crystals. Place lid back onto the container and push down to secure seal.			
9)	Waste Filter (blue)	Stops the crystals from getting into the vacuum pump. Waste filter is located in the waste container.			
10)	Fitting for Crystal Delivery Tube	Fitting where Crystal Delivery Tube is connected.			
11)	Elbow for Crystal Recovery Tube	Elbow where Crystal Recovery Tube is connected.			
12)	Crystal Delivery Tube (small tube)	One end of the tubing connects to the fitting and the other end connects to the handpiece.			

**To order the above parts, call UBS Aesthetics at  
1-800-925-5022.**